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## MedM HealthApp

The collage features several screenshots from the MedM HealthApp and a central image of the Betachek C50 glucose meter. The screenshots include:

- A list of readings with timestamps and glucose values (e.g., 4.8 mmol/L, 5.1 mmol/L, 5.0 mmol/L, 6.2 mmol/L).
- A summary screen for the period 16 May - 23 May showing 73% Normal Glucose, 19% High Glucose, and 8% Low Glucose.
- A line graph showing glucose levels over a week (26 Apr 2023 - 3 May 2023).
- A detailed view of a reading from 2 May 2023 at 4:15 pm, showing a blood glucose level of 6.2 mmol/L, meal information (Before meal), and a scale based on ADA Diabetes Guidelines.
- An export menu for blood glucose data, allowing selection of time intervals (Week, Month, 3 Months, Custom) and formats (Generic CSV, Excel CSV).

The central image shows the Betachek C50 glucose meter with a digital display showing "90 tests".

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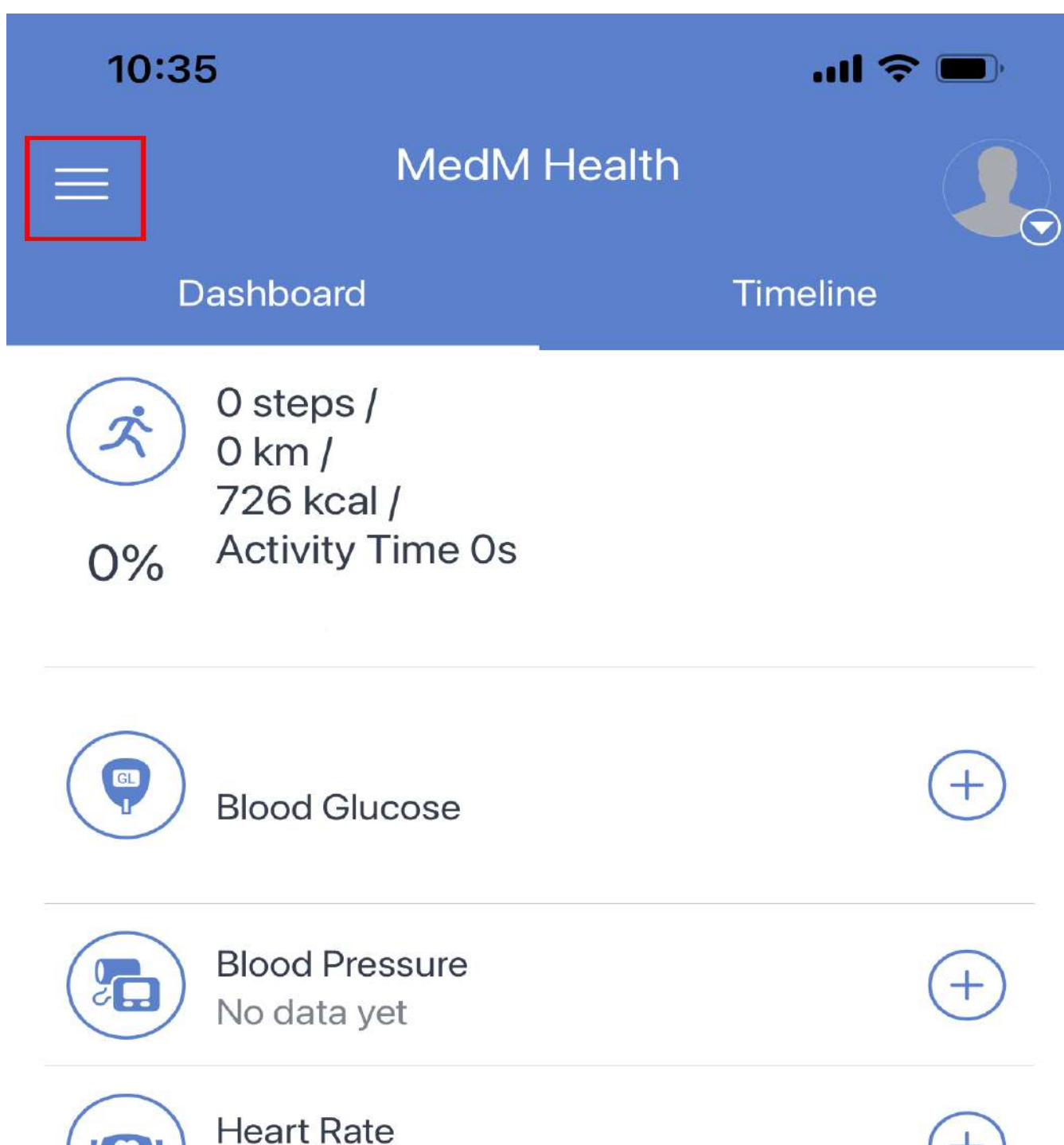
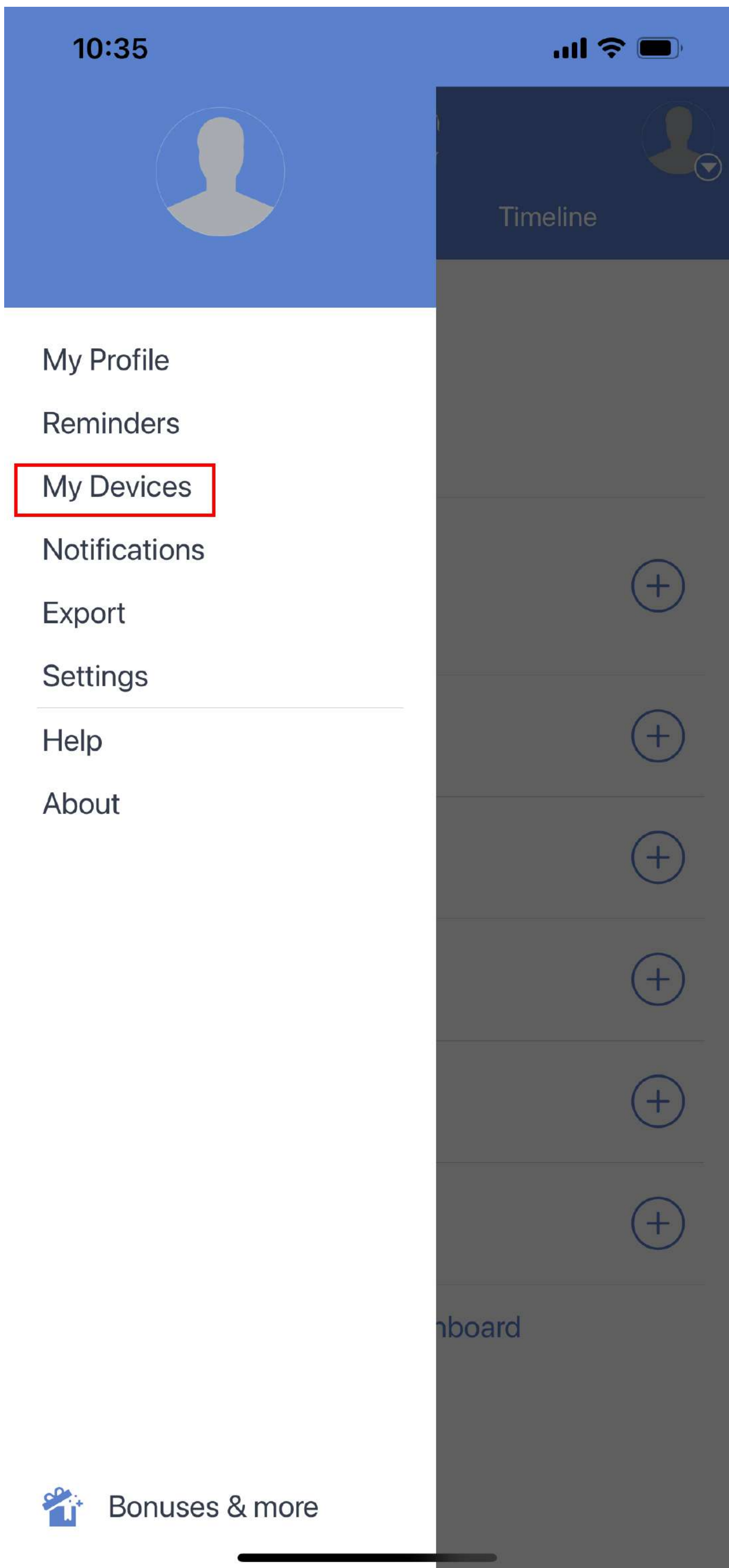
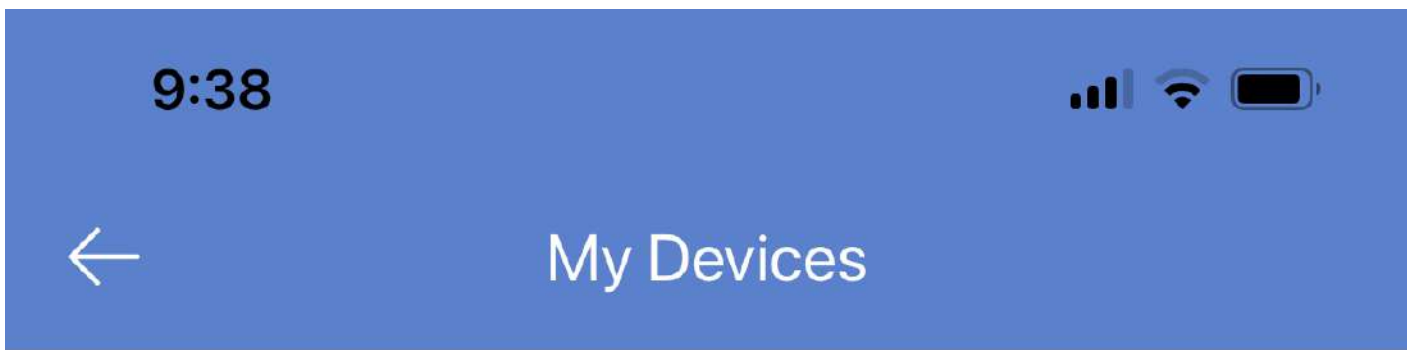
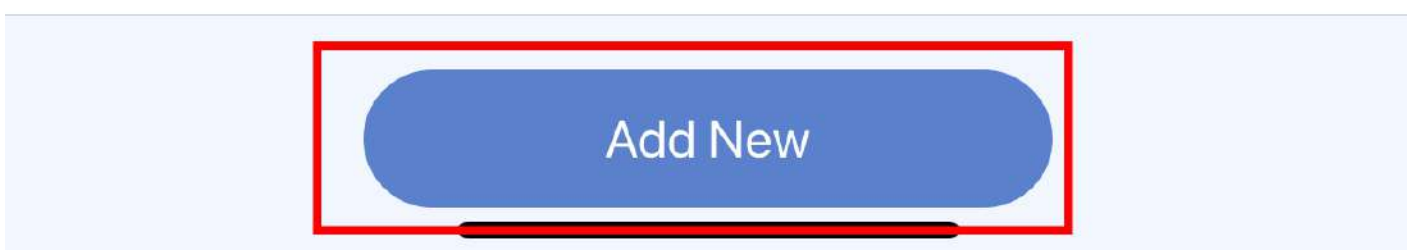
You have no devices.  
To add a device tap 'Add New' button.

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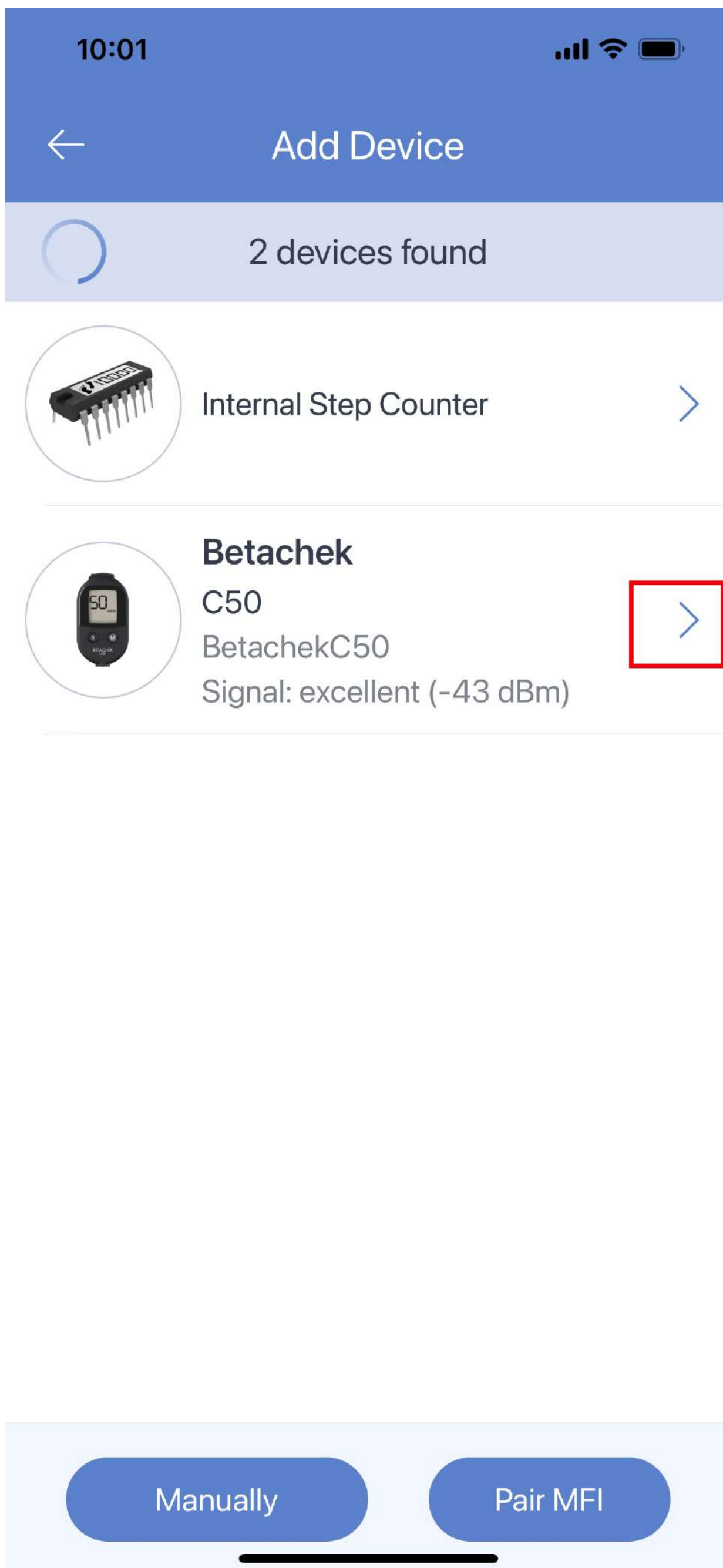
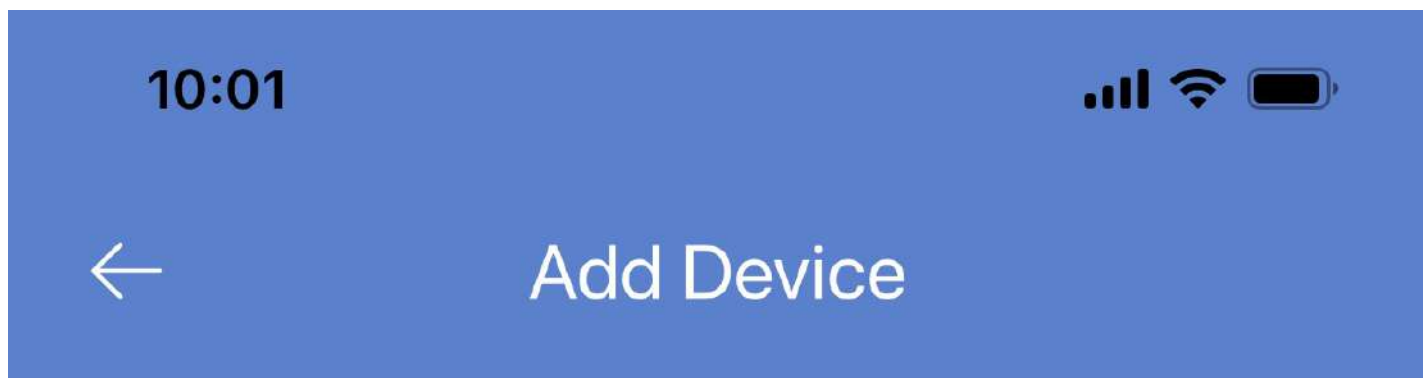
**Betachek**  
C50

Receive data automatically

App has to be in the foreground with screen unlocked.

Receive data in background

Collect data in background or if screen is locked.

**Add to My Devices**

10:01

MedM Health

Dashboard

Timeline

Blood Glucose  
No data yet

MedM Health

Betachek C50 is successfully  
paired with MedM Health.

Make a measurement to upload  
data.

Ok

Manage Dashboard

10:01

MedM Health

Dashboard

Timeline

Blood Glucose  
No data yet

Blood Pressure  
No data yet

Heart Rate  
No data yet

**Bluetooth Pairing Request**  
"BetachekC50" would like to pair with  
your iPhone.

Cancel

Pair

Temperature  
No data yet

Weight  
No data yet

Manage Dashboard

**Brauchen Sie Hilfe?**  
**E-Mail: [support@betachek.com](mailto:support@betachek.com)**